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## For better X-rays

Thoracic spine, lumbar spine, shoulders, pelvis, etc. demand sophisticated X-ray technology on account of the high subject contrast circumference involved.

Equipped with 2 elements, the "Imatec" universal pre-emphasis filter produces pictures which disclose information hitherto obscured by the maximum blackening of the X-ray film.

### 1) Some sample applications Thoracic spine A-P

starting from the head, adjust 1 filter element over 2/3 of the thoracic spine

### Dorsal spine lat.

starting from the head, adjust filter element over 2/3 of the dorsal spine

### Shoulder

allow 1 filter element to impinge diagonally sideways over 1/3 of the photographic field

### Pelvis

slide 1 filter element from each side up to 1/3 into the photographic field

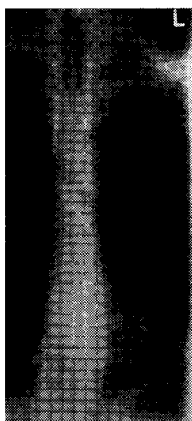
### Feet and hands A-P

slide 1 filter element as far as desired into the photographic field



### 2) Positioning and fixing the filter elements to the collimator

Slide mounting plate into the accessories rail of the X-ray apparatus (please quote dimensions when ordering).

The filter elements can be slid into any position singly or in pairs with the aid of their retaining magnets. The filter effect may be checked at any time in the photographic field of the collimator.

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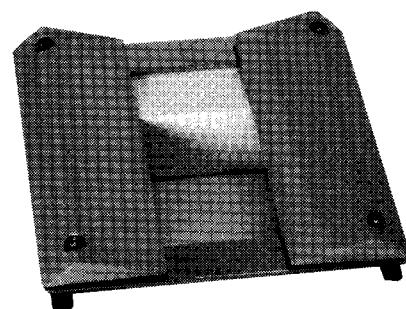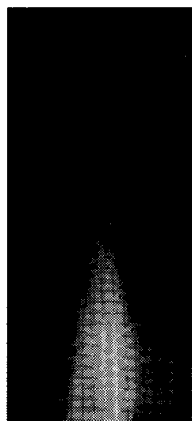We reserve the right to make technical changes.

08.98